

TIM TRIO

Procedure for saving study to CD
or DVD

Procedure

- The procedure for saving a study or a sequence of a study on to a CD or DVD is simple. It is more common to save selected sequences and therefore a CD is sufficient. However, if you plan to save an entire study a DVD would provide more storage space.
- The following pages show you how to save on to a DVD or CD. The procedure is slightly different from the previous TRIO system.

- Insert the CD or DVD in the second drive. The first drive is a read only drive.



Register...	Num 0
Browser...	Num .
Schedule...	
Close Patient	
Save As...	
Filming Layout...	
Film Task Status...	
Film Preview...	
Expose Film Task	Ctrl+P
Copy to Film Sheet	Num Enter
Search	
Search Selected...	

Click "Patient" and select "Browser".



H University of California Davis
TrioTim
MR B15
HFS
+LPH



Exam

Viewing

Filming

3D

Tools	Image	View

Patient	Eval	Exam

A

MF 1.00

TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND

A1
C:PH
*f12d1 / 40

E
TP 0
SP L0.7
SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

10cm

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

H University of California Davis
TrioTim
MR B15
HFS
+LPH



Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Local Database test Warmup warrup

Scheduler dqa-tests-091609

DVD-ROM

DVD-R

[3] Design
foom/1111 / INP/LOC

[2] act_epi2d_bold
foom/1111 / INP/LOC

[1] localizer
foom/1111 / INP/LOC

Select the sequence that you want to save to CD or DVD.

A

Patient name test Date of birth 4/4/1987 Patient ID 111
Study description Warmup warrup Study date 9/17/2009 Modality MR

1 L0.7 2 A57.6 3 F1.4

Current Filter: Off

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND
A1
C:PH
*f12d1 / 40

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

Tools Image View

Patient Eval Exam

Exam

Viewing

Filming

3D

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

H University of California Davis
TrioTim
MR B15
HFS
+LPH



Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Import Ctrl+I
Send to leo1629 Num +
Send to...
Export to...
Eject from DVD-R
Eject from...
Eject Finalized from DVD-R
Local Job Status... Ctrl+L
Network Job Status... Ctrl+N
Import from Off-line...
Export to Off-line...
Record Off-line Files

Local Database
Scheduler
DVD-ROM
DVD-R

Warmup warmup [6] act_epi2d_bold
foom/1111 / INP/LOC
[99] PhoenixZIPReport
foom/1111 / INP/LOC
[3] Design
foom/1111 / INP/LOC
[2] act_epi2d_bold
foom/1111 / INP/LOC
[4] localizer

Exam

Viewing

Filming

3D

A

Click Transfer in the "Browser" and select "Export to".

Patient name test Date of birth 4/4/1987 Patient ID 111

Study Descri...	Study Date and ...	Series	Work Status	Mark Status	Requesting Physi...
Warmup warmup	9/17/2009 8:44:37	AI 5	111111	/	

Current Filter: Off

Tools Image View

Patient Eval Exam

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND
A1
C:PH
*f12d1 / 40

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

University of California Davis
TrioTim
MR B15
HFS
+LPH



Exam

Viewing

Filming

3D

Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Export To

Destination	Label on medium	Medium Capacity	Total Capacity
DVD-R		665.30 MB	700.30 MB

Export Cancel Help

"Export to" pop-up appears. Click on "Export".

A

DVD-R

test
///// OPEN

Warmup
///// OPEN

[2] act_eprizd_bold
/com/**** / INP/LOC

[1] localizer
/com/**** / INP/LOC

Patient name test Date of birth 4/4/1987 Patient ID 111

Study Descri...	Study Date and ...	Series	Work Status	Mark Status	Requesting Physi...
Warmup warmup	9/17/2009 8:44:37	AI 5	/////	/	

Current Filter: Off

Tools Image View

auto

Patient Eval Exam

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND
A1
C:PH
*f12d1 / 40

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

University of California Davis
TrioTim
MR B15
HFS
+LPH



Exam

Viewing

Filming

3D

Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Enter Label

Device: DVD-R

Label on medium: TEST

Viewing tool:

OK Cancel Help

- /com/1111 / INP/LOC
- [3] Design /com/1111 / INP/LOC
- [2] act_epi2d_bold /com/1111 / INP/LOC
- [1] localizer /com/1111 / INP/LOC

"Enter Label" pop-up appears. Enter name in "Label on medium" field and click viewing tool. Click "OK".

A

Patient name test Date of birth 4/4/1987 Patient ID 111
Study description Warmup warmup Study date 9/17/2009 Modality MR

1 L0.7

2 A57.6

3 F1.4

Tools Image View

auto

Patient Eval Exam

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND

A1
C:PH
*f12d1 / 40

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

H University of California Davis
TrioTim
MR B15
HFS
+LPH



Exam

Viewing

Filming

3D

Patient Browser

Patient: Applications: Transfer Edit View Filter Evaluation Sort Options Help

Local Database

- test (OPEN)
- Scheduler
- DVD-ROM
- DVD-R
- Warmup warr (OPEN)
- dqa-tests-091609 (OPEN)
- tcprot091609 (OPEN)
- test (OPEN)
- Warmup (OPEN)

[3] Design
foom/1111 / INP/LOC

[2] act_epi2d_bold
foom/1111 / INP/LOC

[1] localizer
foom/1111 / INP/LOC

Patient name test Date of birth 4/4/1987 Patient ID 111
Study description Warmup warrup Study date 9/17/2009 Modality MR

1 L0.7 2 A57.6 3 F1.4

Tools Image View

Patient Eval Exam

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND
A1
C:PH
*f12d1 / 40

Record icon appears. Click on icon to see progress of recording.

256*512 |
Sag
W 2600
C 1350



test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

H University of California Davis
TrioTim
MR B15
HFS
+LPH



Exam
Viewing
Filming
3D

Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Recording on DVD-R

Your CD or DVD is currently being recorded.



Remaining time: 00 Mins 14 Secs

OK

- [3] Design
- [2] act_epi2d_bold
- [1] localizer

Progress pop-up appears.

Patient name test Date of birth 4/4/1987 Patient ID 111
Study description Warmup warmup Study date 9/17/2009 Modality MR




1 L0.7 2 A57.6 3 F1.4



Current Filter: Off

Tools Image View

Patient Eval Exam

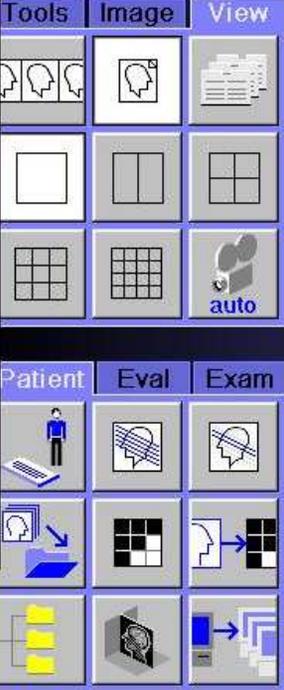

MF 1.00
TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND
A1
C:PH
*f12d1 / 40

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

test
111
*4/4/1987, M, 22Y
STUDY 1
9/17/2009
8:47:50 AM
1 IMA 1 / 3

H University of California Davis
TrioTim
MR B15
HFS
+LPH



Patient Browser

Patient Applications Transfer Edit View Filter Evaluation Sort Options Help

Import Ctrl+I
Send to leo1629 Num +
Send to...
Export to...
Eject from DVD-R
Eject from...
Eject Finalized from DVD-R
Local Job Status... Ctrl+L
Network Job Status... Ctrl+N
Import from Off-line...
Export to Off-line...
Record Off-line Files

Local Database
Scheduler
DVD-ROM
DVD-R

Warmup

[3] Design
[2] act_epi2d_bold
[1] localizer

Exam

Viewing

Filming

3D

When recording has completed. Click on "Transfer" in "Browser". Select "Eject from DVD-R". CD or DVD will eject. Do not use the eject button on the DVD drive, it will not eject your CD or DVD.

1 L0.7
2 A57.6
3 F1.4

MF 1.00

TR 20.0
TE 5.0
TA 02.55*3
BW 180.0
M/ND

A1
C:PH
*f12d1 / 40

Current Filter: Off

SL 10.0
FoV 280*280
256*512 I
Sag
W 2600
C 1350

Tools Image View

Patient Eval Exam