

ZOLL R Series ALS Skills Checklist #DAHS-NSCRSALS17

Page 1 of 3

Name:	Employee ID #:
Unit:	Title:

PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.

These skills will be considered complete when all below performance criteria are completed and pages 1, 2, and 3 are scanned and emailed to hs-cppn@ucdavis.edu

References:	Date Completed (or N/A)	Verifier Initials
UC Davis Health Policy 6005: Automated External Defibrillator (AED-Zoll)		
Completed the assigned ZOLL R Series ALS Defibrillator tutorials in UC Learning		
TEST MODE		
Successfully demonstrates 30 Joule defibrillator test		
Can check and change paper		
AED/MANUAL MODE		
Can turn on device and convert from AED to manual mode		
MONITOR MODE		
Applies 3-lead or 12-lead ECG		
Locates Recorder key and prints a strip		
Access HR menu and demonstrate how to change settings		
Locate NIBP soft key and activate manual BP measurement		
Access NIBP menu and verbalize options		
Demonstrate how to change NIBP alarm settings		
Change NIBP mode from Manual to Automatic		
Change Automatic mode intervals		
Access SpO2 menu and verbalize options		
Access CO2 menu and verbalize options		
MANUAL DEFIBRILLATION		
Locates multifunction cable		
Confirms shockable rhythm		
Selects defibrillator mode (red)		
Presses Energy Select or Charge button		
Tells everyone to stand clear		
Delivers shock at desired energy level		
Defines and adjusts energy levels for adults (120,150, 200J) and Pediatrics (2-4J/kg)		

ZOLL R Series ALS Skills Checklist #DAHS-NSCRSALS17	
Page 2 of 3	
Name:	PPS#:
Unit:	Title:
PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.	
These skills will be considered complete when all below performance criteria are completed and pages 1, 2, and 3 are scanned and emailed to hs-cppn@ucdavis.edu	

CPR FEEDBACK	Date Completed (or N/A)	Verifier Initials
Demonstrates steps to fill CPR Index™ – understands proper rate/depth		
Shows that if rate is too slow, metronome beeps and Rate prompt appears		
Speeds up to silence metronome and allow the Rate prompt to disappear		
Shows that if depth is too shallow, the Depth prompt appears on the screen		
Pushes hard to allow Depth prompt to disappear		
Demonstrates understanding of See-Thru CPR® filtered ECG		
SYNCHRONIZED CARDIOVERSION		
Puts device into SYNC mode		
Selects desired energy		
Presses charge button		
Tells everyone to stand clear		
Delivers synchronized shock		
States and demonstrates that SYNC must be activated for each and every synchronous cardioversion		
PACING		
Turns up pacing output (mA) until capture is achieved – identifies capture		
Adjusts pace rate		
Understands pausing for visualizing patients underlying rhythm		
PADS		
Connects OneStep™ pads to OneStep cable (or other pads/paddles if applicable)		
Opens OneStep packaging correctly		
Demonstrates proper pad placement for defibrillation, pacing, and cardioversion		
Identifies CPR Sensor and explains its purpose		
INTERNAL PADDLES		
Understands how to connect internal paddles to OneStep™ cable		
Selects defibrillator mode (red)		
Understands 10J default energy level with range of 1 to 50 Joules		

ZOLL R Series ALS Skills Checklist #DAHS-NSCRSALS17

Page 3 of 3

Name:	PPS#:
Unit:	Title:
PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.	
These skills will be considered complete when all below performance criteria are completed and pages 1, 2, and 3 are scanned and emailed to hs-cppn@ucdavis.edu	

SUPERUSER/TRAINER	Date Completed (or N/A)	Verifier Initials
Demonstrate how to use additional options (Mentor mode, Set the clock, etc.)		
Understands how to change parameter settings (NIBP, EtCO2, SpO2)		
Understands purpose of Code Marker		
Can access data from the code (Print Chart, Print Log, or Transfer Data).		
User demonstrates sufficient understanding of device to train other users in its use.		

Preceptor Signature: Signature and Printed Name of Preceptor or other verified personnel who have initialed on this form:		
Initial:	Print Name:	Signature:

PRECEPTEE STATEMENT AND SIGNATURE:

I have read and understand the appropriate UC Davis Health Policies/Procedures and/or equipment operations manual, I have demonstrated the ability to perform the verified skills as noted, and I have the knowledge of the resources available to answer questions.

Printed Name	Signature	Date
---------------------	------------------	-------------